Instructions for Use

sTfR ELISA

IVD

REF EIA-5925

Σ 96

Distributed by:

**Please use only the valid version of the Instructions for Use provided with the kit.
Verwenden Sie nur die jeweils gültige, im Testkit enthaltene, Gebrauchsanweisung.
Si prega di usare la versione valida delle istruzioni per l'uso a disposizione con il kit.
Por favor, se usa solo la version valida de la metodico técnico incluido aqui en el kit.
Utilisez seulement la version valide des Instructions d'utilisation fournies avec le kit.**

Table of Contents / Inhaltsverzeichnis / Tabella die Contenuti / Tabla de Contenidos / Sommaire

1	INTRODUCTION.....	2	1	INTRODUCCIÓN.....	26
2	PRINCIPLE OF THE TEST.....	2	2	FUNDAMENTO DEL ENSAYO.....	26
3	WARNINGS AND PRECAUTIONS.....	3	3	PRECAUCIONES.....	26
4	REAGENTS.....	4	4	COMPONENTES DEL KIT.....	27
5	SPECIMEN COLLECTION AND PREPARATION.....	6	5	MUESTRAS.....	29
6	ASSAY PROCEDURE.....	6	6	PROCEDIMIENTO DE ENSAYO.....	29
7	EXPECTED NORMAL VALUES.....	8	7	VALORES ESPERADOS.....	31
8	QUALITY CONTROL.....	8	8	CONTROL DE CALIDAD.....	31
9	PERFORMANCE CHARACTERISTICS.....	9	9	CARACTERÍSTICAS DEL ENSAYO.....	31
10	LIMITATIONS OF USE.....	10	10	LIMITACIONES DE USO.....	32
11	LEGAL ASPECTS.....	11	11	ASPECTOS LEGALES.....	32
1	EINLEITUNG.....	12	1	INTRODUCTION.....	33
2	TESTPRINZIP.....	12	2	PRINCIPE DU TEST.....	33
3	VORSICHTSMAßNAHMEN.....	12	3	PRECAUTIONS D'UTILISATION.....	33
4	BESTANDTEILE DES KITS.....	13	4	COMPOSITION DU KIT.....	34
5	PROBENVORBEREITUNG.....	15	5	ECHANTILLON.....	36
6	TESTDURCHFÜHRUNG.....	15	6	RÉALISATION DU TEST.....	36
7	ERWARTETE WERTE.....	17	7	VALEURS ATTENDUES.....	38
8	QUALITÄTSKONTROLLE.....	17	8	CONTROLE DE QUALITE.....	38
9	ASSAY-CHARACTERISTIKA.....	17	9	CARACTERISTIQUES DU TEST.....	38
10	GRENZEN DES TESTS.....	18	10	LIMITES D'UTILISATION.....	39
11	RECHTLICHE GRUNDLAGEN.....	18	11	ASPECTS LEGAUX.....	39
1	INTRODUZIONE.....	19	12	REFERENCES / LITERATURE.....	40
2	PRINCIPIO DEL TEST.....	19		SYMBOLS USED.....	41
3	PRECAUZIONI.....	19			
4	COMPONENTI DEL KIT.....	20			
5	CAMPIONI.....	22			
6	ATTUAZIONE DEL TEST.....	22			
7	VALORI NORMALI.....	24			
8	CONTROLLO QUALITÀ.....	24			
9	CARATTERISTICHE DEL TEST.....	24			
10	LIMITAZIONE DEL TEST.....	25			
11	ASPETTI LEGALI.....	25			

1 INTRODUCTION

1.1 Intended Use

The **DRG sTfR ELISA** is an enzyme immunoassay for the quantitative *in vitro diagnostic* measurement of sTfR (soluble Transferrin Receptor) in serum or plasma (EDTA-, heparin- or citrate plasma)

1.2 Summary and Explanation

The Transferrin Receptor (TfR) is localized primarily on erythroid progenitor cells. It is a dimeric transmembrane protein composed of two identical subunits. TfR transports iron into the cells by binding and internalization of diferric transferrin (1-4). Soluble transferrin receptor (sTfR) is generated by specific proteolytic cleavage at an extracellular site of TfR and is constantly shedded as monomer of 71.7 kD into the blood (5,6). The serum concentration of sTfR reflects the amount of the cellular receptor (1,7). sTfR concentration in serum increases with hyperproliferative erythropoiesis (increase of erythroblasts and reticulocytes in iron deficiency) and ineffective or hyperplastic erythropoiesis (e.g. β -thalassemia, polycythemia, myelodysplastic syndrome or hemolytic anemia). Hence, the levels of sTfR reflect the receptor density on cells (tissue iron status) and the number of cells with receptors (erythropoietic activity). sTfR level is elevated in iron deficiency, but not in anemia associated with chronic disease (8-11). sTfR level is reduced in hypoplastic erythropoiesis (e.g. anemia of chronic disease, chronic renal failure, aplastic anemia or post-transplant anemia) (12,13). Therefore, measurement of sTfR is particularly useful, when trying to distinguish between iron deficient individuals and anemic individuals with chronic disease (14-17).

2 PRINCIPLE OF THE TEST

The DRG sTfR ELISA is a solid phase enzyme-linked immunosorbent assay (ELISA) based on the **sandwich principle**. The microtiter wells are coated with a monoclonal (mouse) antibody directed towards a unique antigenic site of the sTfR molecule. An aliquot of patient sample containing endogenous sTfR is incubated in the coated well with enzyme conjugate, which is an anti-sTfR antibody conjugated with horseradish peroxidase. After incubation the unbound conjugate is washed off.

The amount of bound peroxidase conjugate is proportional to the concentration of sTfR in the sample.

Having added the substrate solution, the intensity of colour developed is proportional to the concentration of sTfR in the patient sample.

3 WARNINGS AND PRECAUTIONS

1. This kit is for in vitro diagnostic use only. For professional use only.
2. All reagents of this test kit which contain human serum or plasma have been tested and confirmed negative for HIV I/II, HBsAg and HCV by FDA approved procedures. All reagents, however, should be treated as potential biohazards in use and for disposal.
3. Before starting the assay, read the instructions completely and carefully. Use the valid version of instructions for use provided with the kit. Be sure that everything is understood.
4. The microplate contains snap-off strips. Unused wells must be stored at 2 °C to 8 °C in the sealed foil pouch and used in the frame provided.
5. Pipetting of samples and reagents must be done as quickly as possible and in the same sequence for each step.
6. Use reservoirs only for single reagents. This especially applies to the substrate reservoirs. Using a reservoir for dispensing a substrate solution that had previously been used for the conjugate solution may turn solution colored. Do not pour reagents back into vials as reagent contamination may occur.
7. Mix the contents of the microplate wells thoroughly to ensure good test results. Do not reuse microwells.
8. Do not let wells dry during assay; add reagents immediately after completing the rinsing steps.
9. Allow the reagents to reach room temperature (21 °C to 26 °C) before starting the test. Temperature will affect the absorbance readings of the assay. However, values for the patient samples will not be affected.
10. Never pipet by mouth and avoid contact of reagents and specimens with skin and mucous membranes.
11. Do not smoke, eat, drink or apply cosmetics in areas where specimens or kit reagents are handled.
12. Wear disposable latex gloves when handling specimens and reagents. Microbial contamination of reagents or specimens may give false results.
13. Handling should be done in accordance with the procedures defined by an appropriate national biohazard safety guideline or regulation.
14. Do not use reagents beyond expiry date as shown on the kit labels.
15. All indicated volumes have to be performed according to the protocol. Optimal test results are only obtained when using calibrated pipettes and microtiter plate readers.
16. Do not mix or use components from kits with different lot numbers. It is advised not to exchange wells of different plates even of the same lot. The kits may have been shipped or stored under different conditions and the binding characteristics of the plates may result slightly different.
17. Avoid contact with Stop Solution containing 0.5 M H₂SO₄. It may cause skin irritation and burns.
18. Some reagents contain Proclin 300, BND and/or MIT as preservatives. In case of contact with eyes or skin, flush immediately with water.
19. TMB substrate has an irritant effect on skin and mucosa. In case of possible contact, wash eyes with an abundant volume of water and skin with soap and abundant water. Wash contaminated objects before reusing them. If inhaled, take the person to open air.
20. Chemicals and prepared or used reagents have to be treated as hazardous waste according to the national biohazard safety guideline or regulation.
21. For information on hazardous substances included in the kit please refer to Safety Data Sheets. Safety Data Sheets for this product are available upon request directly from DRG.

4 REAGENTS

4.1 Reagents provided

1. **Microtiterwells**, 12 x 8 (break apart) strips, 96 wells;
Wells coated with anti-sTfR antibody (monoclonal).
2. **Standard (Standard 0 - 5)**, 6 vials, 1 mL each, lyophilized;
Concentrations: 0 – 10 – 20 – 40 – 80 – 160 ng/mL
Conversion: 1 ng/mL = 13.96 pmol/L
*The standards are calibrated against the following reference material:
WHO Reference Reagent, NIBSC code: 07/202*
See “Reagent Preparation”.
Contain non-mercury preservative.
3. **Control Low & High**, 2 vials, 1 mL each, lyophilized;
For control values and ranges please refer to vial label or QC-Datasheet.
See “Reagent Preparation”.
Contain non-mercury preservative.
4. **Sample Diluent**, 1 vial, 50 mL, ready to use;
Contains non-mercury preservative.
5. **Enzyme Conjugate**, 1 vial, 14 mL, ready to use;
Anti-sTfR antibody conjugated with horseradish peroxidase;
Contains non-mercury preservative.
6. **Substrate Solution**, 1 vial, 14 mL, ready to use;
Tetramethylbenzidine (TMB).
7. **Stop Solution**, 1 vial, 14 mL, ready to use;
Contains 0.5 M H₂SO₄,
Avoid contact with the stop solution. It may cause skin irritations and burns.
8. **Wash Solution**, 1 vial, 30 mL (40X concentrated);
See “Reagent Preparation”.

4.2 Materials required but not provided

- A microtiter plate calibrated reader (450 ± 10 nm) (e.g. the DRG Instruments Microtiter Plate Reader).
- Calibrated variable precision micropipettes.
- Absorbent paper.
- Distilled or deionized water
- Timer
- Graph paper or software for data reduction
- Tubes for sample dilution

4.3 Storage Conditions

When stored at 2 °C to 8 °C unopened reagents will retain reactivity until expiration date. Do not use reagents beyond this date. Opened reagents must be stored at 2 °C to 8 °C. Microtiter wells must be stored at 2 °C to 8 °C. Once the foil bag has been opened, care should be taken to close it tightly again.
Opened kits retain activity for 8 weeks if stored as described above.

4.4 Reagent Preparation

Bring all reagents and required number of strips to room temperature (20 °C to 25 °C) prior to use.

Standards

Reconstitute the lyophilized contents of each vial with 1 mL deionized water and let stand for at least 10 minutes at room temperature. Mix several times before use.

Note: *The reconstituted standards are stable for 5 days at 2 °C to 8 °C.
For longer storage (up to 8 weeks) aliquot and freeze at -20 °C.*

Control Low & High

Reconstitute the lyophilized content of each vial with 1 mL deionized water and let stand for at least 10 minutes at room temperature. Mix the control several times before use.

Note: *The reconstituted controls are stable for 5 days at 2 °C to 8 °C.
For longer storage (up to 8 weeks) aliquot and freeze at -20 °C.*

Wash Solution

Add deionized water to the 40X concentrated Wash Solution.

Dilute 30 mL of concentrated *Wash Solution* with 1170 mL deionized water to a final volume of 1200 mL.

The diluted Wash Solution is stable for 2 weeks at room temperature.

4.5 Disposal of the Kit

The disposal of the kit must be made according to the national regulations. Special information for this product is given in the Safety Data Sheet, section 13.

4.6 Damaged Test Kits

In case of any severe damage to the test kit or components, DRG has to be informed in writing, at the latest, one week after receiving the kit. Severely damaged single components should not be used for a test run. They have to be stored until a final solution has been found. After this, they should be disposed according to the official regulations.

5 SPECIMEN COLLECTION AND PREPARATION

Serum or plasma (EDTA-, heparin- or citrate plasma) can be used in this assay.

Do not use haemolytic, icteric or lipaemic specimens.

Please note: Samples containing sodium azide should not be used in the assay.

5.1 Specimen Collection

Serum:

Collect blood by venipuncture (e.g. Sarstedt Monovette for serum), allow to clot, and separate serum by centrifugation at room temperature. Do not centrifuge before complete clotting has occurred. Patients receiving anticoagulant therapy may require increased clotting time.

Plasma:

Whole blood should be collected into centrifuge tubes containing anti-coagulant (e.g. Sarstedt Monovette with the appropriate plasma preparation) and centrifuged immediately after collection.

5.2 Specimen Storage and Preparation

Serum samples should be capped and may be stored for up to 5 days at 2 °C to 8 °C prior to assaying.

Plasma samples should be capped and may be stored for up to 1 day at 2 °C to 8 °C prior to assaying.

Specimens (serum and plasma) held for a longer time (up to 12 months) should be frozen only once at -20 °C prior to assay. Thawed samples should be inverted several times prior to testing.

5.3 Specimen Dilution

Prior to the assay, each sample need to be diluted 1:51 in Sample Diluent in a separate vial.

Example of 1:51 dilution:

10 µL sample + 500 µL *Sample Diluent* (mix thoroughly)

For the **calculation of the final concentrations**, results must be multiplied by a dilution factor of **51**.

If in an initial assay, a specimen is found to contain more than the highest standard, the specimen can be further diluted with *Sample Diluent* and re-assayed as described in Assay Procedure.

For the calculation of the concentrations this additional dilution factor has to be taken into account too.

Example:

Dilution 1:2: 100 µL sample (diluted 1:51) + 100 µL *Sample Diluent* (mix thoroughly)

6 ASSAY PROCEDURE

6.1 General Remarks

- All reagents and specimens must be allowed to come to room temperature before use. All reagents must be mixed without foaming.
- Once the test has been started, all steps should be completed without interruption.
- Use new disposal plastic pipette tips for each standard, control or sample in order to avoid cross contamination.
- Absorbance is a function of the incubation time and temperature. Before starting the assay, it is recommended that all reagents are ready, caps removed, all needed wells secured in holder, etc. This will ensure equal elapsed time for each pipetting step without interruption.
- As a general rule the enzymatic reaction is linearly proportional to time and temperature.

6.2 Test Procedure

Each run must include a standard curve.

1. Secure the desired number of Microtiter wells in the frame holder.
2. Dispense **100 µL** of each **Standard, Control** and **diluted sample** with new disposable tips into appropriate wells.
3. Incubate for **30 minutes** at room temperature.
4. Briskly shake out the contents of the wells.
Rinse the wells **3 times** with **400 µL** diluted *Wash Solution* per well, if a plate washer is used - or -
rinse the wells **4 times** with **300 µL** diluted *Wash Solution* per well for manual washing.
Strike the wells sharply on absorbent paper to remove residual droplets.
Important note:
The sensitivity and precision of this assay is markedly influenced by the correct performance of the washing procedure!
5. Dispense **100 µL Enzyme Conjugate** into each well.
6. Incubate for **30 minutes** at room temperature.
7. Briskly shake out the contents of the wells.
Rinse the wells **3 times** with **400 µL** diluted *Wash Solution* per well, if a plate washer is used - or -
rinse the wells **4 times** with **300 µL** diluted *Wash Solution* per well for manual washing.
Strike the wells sharply on absorbent paper to remove residual droplets.
8. Add **100 µL of Substrate Solution** to each well.
9. Incubate for **15 minutes** at room temperature.
10. Stop the enzymatic reaction by adding **100 µL of Stop Solution** to each well.
11. Determine the absorbance (OD) of each well at **450 ± 10 nm** with a microtiter plate reader.
It is recommended that the wells be read **within 10 minutes** after adding the *Stop Solution*.

6.3 Calculation of Results

1. Calculate the average absorbance values for each set of standards, controls and patient samples.
2. Using linear graph paper, construct a standard curve by plotting the mean absorbance obtained from each standard against its concentration with absorbance value on the vertical (Y) axis and concentration on the horizontal (X) axis.
3. Using the mean absorbance value for each sample determine the corresponding concentration from the standard curve.
4. Automated method: The results in the Instructions for Use have been calculated automatically using a 4-Parameter curve fit. (4 Parameter Rodbard or 4 Parameter Marquardt are the preferred methods.) Other data reduction functions may give slightly different results.
5. The concentration of the samples can be read directly from this standard curve. Samples with concentrations higher than that of the highest standard have to be further diluted or reported as > 160 ng/mL. For the calculation of the concentrations this dilution factor has to be taken into account.

6.3.1 Example of Typical Standard Curve

The following data is for demonstration only and **cannot** be used in place of data generations at the time of assay.

	Standard	Optical Units (450 nm)
Standard 0	0 ng/mL	0.04
Standard 1	10 ng/mL	0.56
Standard 2	20 ng/mL	0.86
Standard 3	40 ng/mL	1.26
Standard 4	80 ng/mL	1.65
Standard 5	160 ng/mL	2.09

7 EXPECTED NORMAL VALUES

It is strongly recommended that each laboratory should determine its own normal and abnormal values.

In a study conducted with apparently healthy individuals, using the DRG sTfR ELISA the following data were observed:

Population	n	Mean (µg/mL)	Median (µg/mL)	2.5 th - 97.5 th Percentile (µg/mL)	Range (min. - max.) (µg/mL)
Males and Females	176	1.43	1.33	0.73 - 2.84	0.48 - 3.09

The results alone should not be the only reason for any therapeutic consequences. The results should be correlated to other clinical observations and diagnostic tests.

8 QUALITY CONTROL

Good laboratory practice requires that controls be run with each calibration curve. A statistically significant number of controls should be assayed to establish mean values and acceptable ranges to assure proper performance.

It is recommended to use control samples according to state and federal regulations. The use of control samples is advised to assure the day to day validity of results. Use controls at both normal and pathological levels.

The controls and the corresponding results of the QC-Laboratory are stated in the QC certificate added to the kit. The values and ranges stated on the QC sheet always refer to the current kit lot and should be used for direct comparison of the results.

It is also recommended to make use of national or international Quality Assessment programs in order to ensure the accuracy of the results.

Employ appropriate statistical methods for analysing control values and trends. If the results of the assay do not fit to the established acceptable ranges of control materials patient results should be considered invalid.

In this case, please check the following technical areas: Pipetting and timing devices; photometer, expiration dates of reagents, storage and incubation conditions, aspiration and washing methods.

After checking the above mentioned items without finding any error contact your distributor or DRG directly.

9 PERFORMANCE CHARACTERISTICS

9.1 Assay Dynamic Range

The range of the assay is between 0.145 – 160 ng/mL.

9.2 Specificity of Antibodies (Cross-Reactivity)

The following substances were tested for cross reactivity of the assay:

Substance	Added Conc. ng/mL	Measured Conc. ng/mL	Cross-reactivity (%)
Transferrin human	85 000	0	0
Albumin human	200 000	0	0
Ferritin human	100	0	0

9.3 Sensitivity

The analytical sensitivity of the DRG ELISA was calculated by adding 2 standard deviations to the mean of 20 replicate analyses of the *Standard 0* and was found to be 0.01 ng/mL.

The Limit of Blank (LoB) is 0.008 ng/mL.

The Limit of Detection (LoD) is 0.145 ng/mL.

The Limit of Quantification (LoQ) is 0.478 ng/mL.

9.4 Reproducibility

9.4.1 Intra Assay

The within assay variability was determined by measuring each serum sample 20 times per run:

Sample	n	Mean (µg/mL)	CV (%)
1	20	0.98	4.97
2	20	2.25	5.79
3	20	4.72	5.39

9.4.2 Inter Assay

The between assay variability was determine by measuring each serum sample 2 in 10 independent runs on 6 days:

Sample	n	Mean (µg/mL)	CV (%)
1	20	1.09	5.92
2	20	2.09	2.52
3	20	2.63	4.03
4	20	3.70	3.68

9.4.3 Inter-Lot

The inter-assay (between-lots) variation was determined by measuring each sample 6 times with 3 different kit lots (n = 18):

Sample	n	Mean (µg/mL)	CV (%)
1	18	1.07	11.04
2	18	5.82	8.46
3	18	2.47	9.42

9.5 Recovery

Samples have been spiked by adding sTfR solutions with known concentrations.

The recovery (%) was calculated by multiplying the ratio of measured and expected values with 100.

	Serum 1	Serum 2	Serum 3	EDTA	Li-Heparin	Citrate	
Concentration ($\mu\text{g/mL}$)	1.68	2.18	1.54	1.70	1.55	1.81	
Average Recovery (%)	105.5	108.3	95.6	101.1	98.5	100.5	
Range of Recovery (%)	from	98.3	102.9	89.8	93.4	93.8	94.7
	to	109.8	112.6	103.8	106.7	102.7	104.7

9.6 Linearity

Samples were measured undiluted and in serial dilutions from 1:2 to 1:16 with standard 0. The recovery (%) was calculated by multiplying the ratio of expected and measured values with 100.

	Serum 1	Serum 2	Serum 3	EDTA	Li-Heparin	Citrate	
Concentration ($\mu\text{g/mL}$)	1.32	2.75	4.68	3.27	2.65	2.07	
Average Recovery (%)	93.0	103.7	105.9	101.4	110.6	97.3	
Range of Recovery (%)	from	86.3	96.1	92.5	99.3	108.6	92.7
	to	101.0	114.2	112.5	103.7	112.1	100.8

10 LIMITATIONS OF USE

Reliable and reproducible results will be obtained when the assay procedure is performed with a complete understanding of the package insert instruction and with adherence to good laboratory practice.

Any improper handling of samples or modification of this test might influence the results.

10.1 Interfering Substances

Haemoglobin (up to 4 mg/mL), Bilirubin (up to 0.5 mg/mL) and Triglyceride (up to 7.5 mg/mL) have no influence on the assay results.

10.2 Drug Interferences

Until today no substances (drugs) are known to us, which have an influence to the measurement of sTfR in a sample.

10.3 High-Dose-Hook Effect

Hook effect was not observed in this test up to a concentration of 2170 ng/mL of sTfR (after 1:51 dilution). This corresponds to a final concentration of 110.7 $\mu\text{g/mL}$.

11 LEGAL ASPECTS

11.1 Reliability of Results

The test must be performed exactly as per the manufacturer's instructions for use. Moreover the user must strictly adhere to the rules of GLP (Good Laboratory Practice) or other applicable national standards and/or laws. This is especially relevant for the use of control reagents. It is important to always include, within the test procedure, a sufficient number of controls for validating the accuracy and precision of the test.

The test results are valid only if all controls are within the specified ranges and if all other test parameters are also within the given assay specifications. In case of any doubt or concern please contact DRG.

11.2 Therapeutic Consequences

Therapeutic consequences should never be based on laboratory results alone even if all test results are in agreement with the items as stated under point 11.1. Any laboratory result is only a part of the total clinical picture of a patient.

Only in cases where the laboratory results are in acceptable agreement with the overall clinical picture of the patient should therapeutic consequences be derived.

The test result itself should never be the sole determinant for deriving any therapeutic consequences.

11.3 Liability

Any modification of the test kit and/or exchange or mixture of any components of different lots from one test kit to another could negatively affect the intended results and validity of the overall test. Such modification and/or exchanges invalidate any claim for replacement.

Claims submitted due to customer misinterpretation of laboratory results subject to point 11.2 are also invalid.

Regardless, in the event of any claim, the manufacturer's liability is not to exceed the value of the test kit. Any damage caused to the test kit during transportation is not subject to the liability of the manufacturer.

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12 REFERENCES / LITERATURE

1. Cook JD, Baynes RD and Skikne BS. The physiological significance of circulating transferrin receptors, *Adv Exp Med Biol.* 1994; 352:119-26.
2. Omary MB and Trowbridge IS. Biosynthesis of the human transferrin receptor in cultured cells, *J. Biol. Chem.* 1981; 256:12888.
3. Schneider C et al. Primary structure of human transferrin receptor deduced from the mRNA sequence. *Nature* 1984; 311: 675-678.
4. McClelland A et al. The human transferrin receptor gene: Genomic organization, and the complete primary structure of the receptor deduced for a cDNA sequence, *Cell.* 1984; 39:267.
5. Shih YJ et al. Serum transferrin receptor is a truncated form of tissue receptor, *J. Biol. Chem.* 1990; 265:19077.
6. Baynes RD et al. Production of soluble transferrin receptor by K562 erythroleukemia cells, *Proc.Soc. Exp. Med.* 1993; 204:65.
7. Luck AN and Mason AB. Structure and dynamics of drug carriers and their interaction with cellular receptors: focus on serum transferrin, *Adv Drug Deliv Rev.* 2013; 65(8):1012-1019.
8. Ferguson BJ et al. Serum transferrin receptor distinguishes the anemia of chronic disease from iron deficiency anemia, *J Lab Clin Med.* 1992; 119(4):385-90.
9. Skikne BS, Flowers CH and Cook JD. Serum transferrin receptor: a quantitative measure of tissue iron deficiency, *Blood.* 1990; 1:75(9):1870-6.
10. Braga F et al. Soluble transferrin receptor in complicated anemia, *Clinica Chimica Acta.* 2014; 431: 143-147.
11. Speeckaert MM, Speeckaert R and Delanghe JR. Biological and clinical aspects of soluble transferrin receptor. *Crit Rev Clin Lab Sci.*2010; 47(5-6):213-28.
12. Thorstensen K and Romslo I. The transferrin receptor: its diagnostic value and its potential as therapeutic target, *Scand. J. Lab. Invest.* 1993; 53(Suppl 215):113.
13. Huebers HA et al. Intact transferrin receptors in human plasma and their relation to erythropoiesis, *Blood* 1990; 75:102.
14. Harms K and Kaiser T. Beyond soluble transferrin receptor: old challenges and new horizons, *Best Practice & Research Clinical Endocrinology & Metabolism.* 2015; 29:799-810.
15. Lebron JA. et al. Crystal structure of the hemochromatosis protein HFE and characterization of its interaction with transferrin receptor. *Cell*, 1998; 93, 111.
16. Suchdev PS et al. Assessment of iron status in settings of inflammation: challenges and potential approaches. *m J Clin Nutr.* 2017; 106(Suppl 6):1626S-1633S.
17. Pfeiffer CM, Looker AC. Laboratory methodologies for indicators of iron status: strengths, limitations, and analytical challenges. *Am J Clin Nutr.* 2017; 106(Suppl 6):1606S-1614S.

SYMBOLS USED

Symbol	English	Deutsch	Italiano	Español	Français
	European Conformity	CE-Konformitätskennzeichnung	Conformità europea	Conformidad europea	Conformité normes européennes
	Consult instructions for use *	Gebrauchsanweisung beachten *	Consultare le istruzioni per l'uso	Consulte las instrucciones de uso	Consulter les instructions d'utilisation
	<i>In vitro</i> diagnostic medical device *	<i>In-vitro</i> -Diagnostikum *	Diagnostica in vitro	Diagnóstico in vitro	Diagnostic in vitro
	Catalogue number *	Artikelnummer *	No. di Cat.	No de catálogo	Référence
	Batch code *	Chargencode *	Lotto no	Número de lote	No. de lot
	Contains sufficient for <n> tests *	Ausreichend für <n> Prüfungen *	Contenuto sufficiente per "n" saggi	Contenido suficiente para <n> ensayos	Contenu suffisant pour "n" tests
	Temperature limit *	Temperaturbegrenzung *	Temperatura di conservazione	Temperatura de conservación	Température de conservation
	Use-by date *	Verwendbar bis *	Data di scadenza	Fecha de caducidad	Date limite d'utilisation
	Manufacturer *	Hersteller *	Fabbricante	Fabricante	Fabricant
	Caution *	Achtung *			
	For research use only	Nur für Forschungszwecke	Solo a scopo di ricerca	Sólo para uso en investigación	Seulement dans le cadre de recherches
<i>Distributed by</i>	Distributed by	Vertreiber	Distributore	Distribuidor	Distributeur
<i>Content</i>	Content	Inhalt	Contenuto	Contenido	Conditionnement
<i>Volume/No.</i>	Volume / No.	Volumen / Anzahl	Volume / Quantità	Volumen / Número	Volume / Quantité
<i>Microtiterwells</i>	Microtiterwells	Mikrotiterwells	Micropozzetti	Placas multipocillo	Plaques de micro-titration
<i>Antiserum</i>	Antiserum	Antiserum	Antisiero	Antisero	Antisérum
<i>Enzyme Conjugate</i>	Enzyme Conjugate	Enzymkonjugat	Tracciante enzimatico	Conjugado enzimático	Conjugué enzymatique
<i>Enzyme Complex</i>	Enzyme Complex	Enzymkomplex	Complesso enzimatico	Complejo enzimático	Complexe enzymatique
<i>Substrate Solution</i>	Substrate Solution	Substratlösung	Soluzione di substrato	Solución de sustrato	Solution substrat
<i>Stop Solution</i>	Stop Solution	Stopplösung	Soluzione d'arresto	Solución de parada	Solution d'arrêt
<i>Zero Standard</i>	Zero Standard	Nullstandard	Standard zero	Estándar cero	Zero Standard
<i>Standard</i>	Standard	Standard	Standard	Estándar	Standard
<i>Control</i>	Control	Kontrolle	Controllo	Control	Contrôle
<i>Assay Buffer</i>	Assay Buffer	Assaypuffer	Tampone del test	Tampón de ensayo	Tampon d'essai
<i>Wash Solution</i>	Wash Solution	Waschlösung	Soluzione di lavaggio	Solución de lavado	Solution de lavage
<i>1N NaOH</i>	1N NaOH	1N NaOH	1N NaOH (idrossido di sodio 1N)	1N NaOH	1N NaOH
<i>1 N HCl</i>	1 N HCl	1 N HCl		1 N HCl	1N HCl
<i>Sample Diluent</i>	Sample Diluent	Probenverdünnungs-medium	Diluyente dei campioni	Solución para dilución de la muestra	Solution pour dilution de l'échantillon
<i>Conjugate Diluent</i>	Conjugate Diluent	Konjugatverdünnungs-medium	Diluyente del tracciante	Solución para dilución del conjugado	Solution pour dilution du conjugué